

# Joystick swing away mount Mounting instructions

November 2025



The Swing away mount comes in a version for wheelchairs with quick lock attachment (P, D2, C3, R etc.), and one version for the classic tube attachment (F etc.). It also comes in left/right versions.



1. Assembly the joystick on the top of the swing away mount with two standard hex screws.



2. Attach the joystick mount onto the armrest as normal, making sure the cable runs smoothly on the outside with enough slack to allow the joystick to move freely.

3. Connect the joystick cable, aligning the contacts carefully to not damage any contact pins inside.